



## Independence Solution

**Tunstall**

SOLUTION SHEET

### What is it?

The Tunstall iVi Intelligent Pendant with fall detection works with Tunstall's Vi+ or CEL Personal Emergency Response Systems, replacing the standard personal activator. The iVi features a standard help button with automatic fall detection technology.\*

### How does it work?

By pressing the button on the pendant, a help call is generated through the console unit and sent to the 24-hour Emergency Response Center (ERC). Trained operators can access call details (subscriber's name, address, and relevant information) that are displayed on their screen.

Our ERC operator communicates with the subscriber through the console's loudspeaker and microphone to determine the nature of the call and the appropriate course of action.

Built-in automatic fall detection senses when the subscriber has fallen and triggers a call to the ERC.\* The iVi is the only product of its kind to feature a not-worn alert which will notify our ERC if the iVi has not registered movement for a period of time.\*\*



### Who is it for?

A wide variety of subscribers can benefit from the iVi's intelligent pendant's state-of-the-art technology, including people who:

- Are at high risk for falls
- Have muscle weakness, especially in their legs
- Take medications that may cause dizziness
- Have hypotension (low blood pressure)
- Have neurological conditions such as Parkinson's Disease or Epilepsy
- Have mental, physical, or sensory impairments

The iVi offers family, caregivers, and friends the reassurance that their loved ones can use their iVi pendant to easily call for help whenever they need it from anywhere in and around the home.



## Key Features

- Advanced algorithms
- Adjustable fall sensitivity
- Convenient cancel button
- Supervised replaceable battery
- Optional not worn alert\*\*
- Audible announcements available in English or Spanish
- Small, lightweight design
- Ergonomic alert button
- Compatibility
- 390ft (120m) radio range in free space
- Water resistant (IP67)
- Multiple wearing options
- Supervised auto presence
- FDA registered and ETL listed

## Benefits

- Enables the iVi to distinguish between fall and non-fall events
- Customizable to suit individuals
- Enables the wearer to cancel fall activations if assistance is not required, eliminating unnecessary calls
- Low battery alerts are automatically generated and the replaceable battery allows for easy maintenance
- If wearing compliance is a concern, the iVi can be configured to report if it has not registered movement for a specified period of time
- Subscriber is informed of the call progress and status
- Discreet and easy to carry
- Enables subscribers with visual impairments or limited dexterity to trigger a call
- The iVi can be easily programmed to the Tunstall Vi+ or CEL
- Range suitable for typical dwellings/homes
- Can be worn in shower or bath
- Necklace, brooch, or on a belt
- Notifies the Emergency Response Center if the iVi is not seen within range of the console for a specified period of time
- Tunstall is FDA registered, complies with all medical device regulations, and is ETL listed to conform with UL 1635 and UL 1637 product safety standards.

## Specifications

- |                            |   |
|----------------------------|---|
| • Part Number              | • iVi (66004/09)<br>Console(s): Vi+(57104/570) or CEL (56004/580)                               |
| • Dimensions               | • 2.28" x 1.5" x 0.55" (WxLxD)  |
| • Weight                   | • 0.88oz  |
| • Battery                  | • 12 month battery life • Fully supervised<br>Replaceable battery (Sony CR2450 3v Lithium cell) |
| • Options                  | • Necklace, brooch or belt wearing option Multiple<br>cover color options                       |
| • Maintenance Requirements | • Periodic battery replacement  |
| • Product Compatibility    | • Vi+ and CEL   |

For additional information please call

**1.800.286.2622**

or email [USInfo@tunstall.com](mailto:USInfo@tunstall.com)

36-36 33rd Street, Suite 103, Long Island City, NY 11106

[americas.tunstall.com](http://americas.tunstall.com)

\*Fall detection feature does not detect 100% of falls. If able, users should always press their help button when they need help.

\*\* Not-worn alert is an optional feature and compatible with Tunstall Vi+ and CEL PERS units.